M A X T E R
GLOVE MANUFACTURING SDN BHD
(229862-H)

LOT 6070
Jalan Haji Abdul Manan, 6th Miles Off Jalan Meru
41050 Klang, Selangor, Malaysia
Tel: 603-33929888 (8 lines) Fax: 603-33923328
E-MAIL: info@maxter.com.my

Effective Date: 1st June 2022
Revision No.: Rev.1
IFU-POWDER FREE NITRILE

POWDER FREE NITRILE EXAMINATION GLOVES

INSTRUCTION FOR USE

Specification

Size	XS	S	M	L	XL
Width Tolerance	70-79mm	80-89mm	90-99mm	100-109mm	110 - 119mm
Overall Length	Min 240 mm	Min 240 mm	Min 240 mm	Min 240 mm	Min 240 mm

Main Configuration and Function

The powder free nitrile examination glove is made from synthetic rubber, ambidextrous and non-sterile. It is treated with chlorine which is to facilitate the user in donning the glove and as well as to prevent the glove surface from sticking to each other.

Apply Scope

The powder free nitrile examination glove is a medical device, which protects the hand or part of the hand of the user. The main function of wearing gloves is to protect the wearer against contamination of infectious materials particularly viruses, bacteria, infected blood and body fluids. It is intended for single use only.

The powder free nitrile examination glove is usually used for conducting medical examination, dentistry, clinical examination, diagnostic and therapeutic procedures and also for laboratory purposes.

Usage Method

1. Preparation for use
 - Ensure selection of gloves with correct size.
2. How to use
 - Dry hand before donning the gloves.
 - Take out the product after checking package.
 - Glove need to be donned carefully; do not apply excessive force to pull the gloves when donning. Do not touch against sharp point.
 - Keep the fingernails short and change gloves frequently to avoid tearing issue.

M A X T E R
GLOVE MANUFACTURING SDN BHD
(229862-H)

LOT 6070

Jalan Haji Abdul Manan, 6th Miles Off Jalan Meru
41050 Klang, Selangor, Malaysia
Tel: 603-33929888 (8 lines) Fax: 603-33923328
E-MAIL: info@maxter.com.my

Effective Date: 1st June 2022

Revision No.: Rev.1

IFU-POWDER FREE NITRILE

Attention Proceeding

1. The product is for single use.
2. The gloves should be discarded with an appropriate waste disposal container after use according to local disposal regulations.

Valid

The shelf life of the gloves is 5 years from the date of manufacture.

Storage and Transportation

1. The atmosphere must always be conducive to the storage of the gloves.
2. Gloves before and after opened for use shall be stored in dark, cool and dry place.
3. Gloves need to avoid area with excessive heat. The storage areas need to be controlled below 40°C/ 104 °F.
4. Gloves need to be stored at the areas with good air ventilation.
5. Gloves need to be shielded from direct sunlight and fluorescent lighting.
6. Gloves need to be stored far away from any energy radiating and heat generating equipment.

Manufacturer / Legal Manufacturer:
Maxter Glove Manufacturing Sdn. Bhd.
Lot 6070 Jalan Haji Abdul Manan,
6th Miles Off Jalan Meru,
41050 Klang, Selangor, Malaysia
Tel: 603-33929888
Fax: 603-33923328

Single Registration Number (SRN): **MY-MF-000016719**

Our Authorized Representative in Europe:
OBELIS S.A.
Bd. General Wahis 53,
1030 Brussels,
Belgium.

Single Registration Number (SRN): **BE-AR-000000106**

OBELIS UK LTD
Registered Address:
Sandford Gate
East Point Business Park
Oxford
OX4 6LB
United Kingdom

M A X T E R
GLOVE MANUFACTURING SDN BHD
(229862-H)

LOT 6070
Jalan Haji Abdul Manan, 6th Miles Off Jalan Meru
41050 Klang, Selangor, Malaysia
Tel: 603-33929888 (8 lines) Fax: 603-33923328
E-MAIL: info@maxter.com.my

Effective Date: 1st June 2022
Revision No.: Rev.1
IFU-POWDER FREE NITRILE

Index of CE Symbols

	Manufacturer / Producer		Production date / Date of manufacture
	Expired date / Best before		Batch code / Lot number
	Prohibition on re-use		Mark of conformity to the basic requirements of EU Regulation
XS, S, M, L, XL	Size device		Non-sterile
	Medical device		Food safe
	AQL 1.5		Do not contains or presence of natural rubber latex
	PAP 21		

M A X T E R
GLOVE MANUFACTURING SDN BHD
(229862-H)

LOT 6070

Jalan Haji Abdul Manan, 6th Miles Off Jalan Meru
41050 Klang, Selangor, Malaysia
Tel: 603-33929888 (8 lines) Fax: 603-33923328
E-MAIL: info@maxter.com.my

Effective Date: 1st June 2022
Revision No.: Rev.1
IFU-POWDER FREE NITRILE

REVISION HISTORY			
REVISION	DATE OF RELEASE FOR APPROVAL	MODIFICATION	AUTHOR
0	8 February 2021	Initial Release	Jasleen Chia
1	1 June 2022	- Added in the revision no. - Included symbols of food safe, AQL1.5, PAP21 and do not contain of NRL.	Jasleen Chia

M A X T E R
GLOVE MANUFACTURING SDN BHD
(229862-H)

LOT 6070
Jalan Haji Abdul Manan, 6th Miles Off Jalan Meru
41050 Klang, Selangor, Malaysia
Tel: 603-33929888 (8 lines) Fax: 603-33923328
E-MAIL: info@maxter.com.my

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